

eeZee Mouse PRO


Quick Start

4 Easy Steps



www.lazetek.com

1 Connect Device*

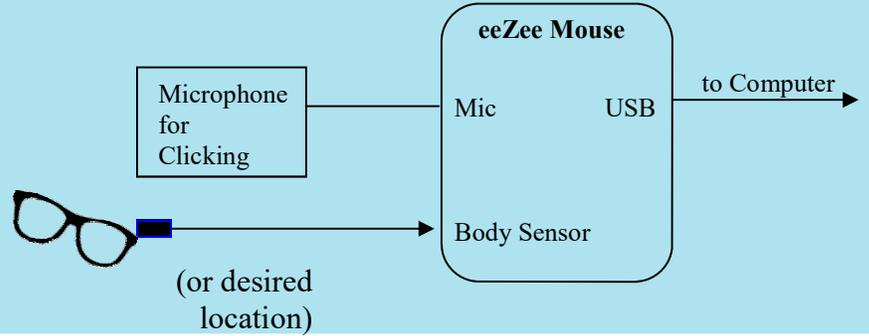


* See back of sheet for additional info

2 Position Body Sensor



* Put on glasses with sensor attached or position sensor as desired

* Sit comfortably, looking at screen

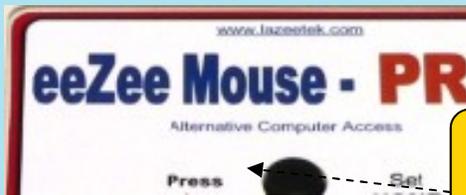

Press "Home"

3 You're Ready!

Tilt to move

Proportional Speed

- Tilt slightly to move SLOWLY
- Tilt farther to move Faster

Rotation

of head does NOT move cursor.

Move Left



Tilt Left
(Lateral tilt)

Move Up

Tilt Head Up
(Pull joystick back)

Move Down

Tilt Head Down
(Push joystick forward)

Stop

Return to HOME position
(release joystick)


Move Right



Tilt Right
(Lateral tilt)

4 Clicking w/ Microphone

Quick sound

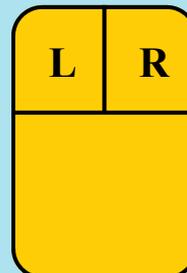


Long sound (~1 Sec)

Drag =

Extended sound

* See back of sheet for additional info

eeZee Mouse PRO - Controls

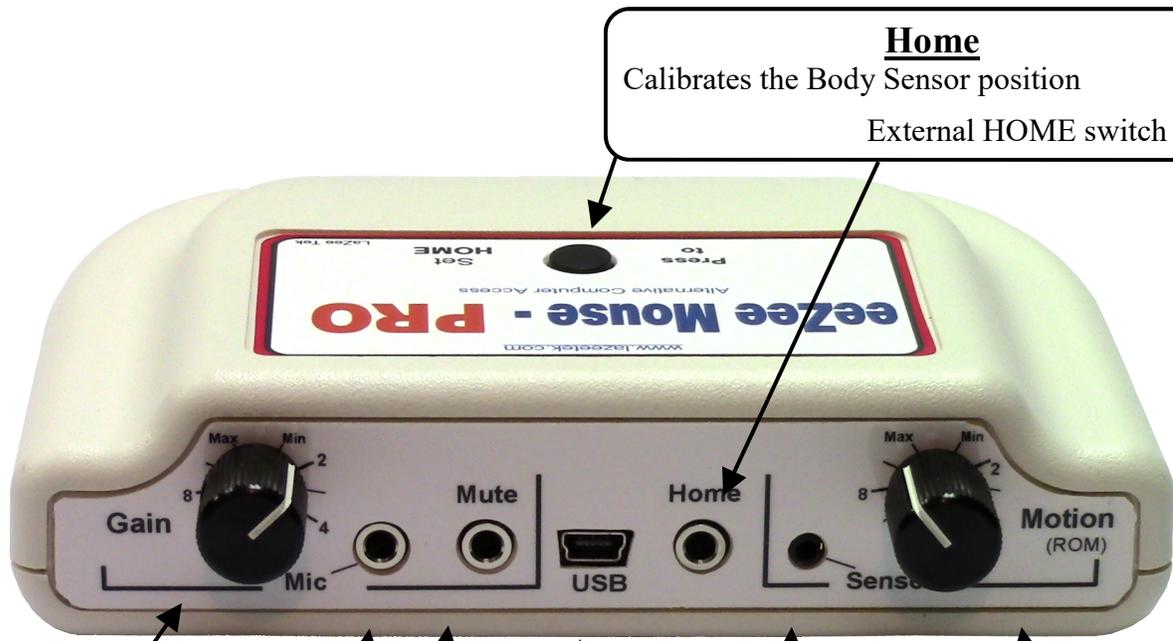

Home
Calibrates the Body Sensor position
External HOME switch input

Mic Gain adjust

Mic Input

Mute
disables microphone

USB to Computer

Body Sensor Input

Activate HOME after all ROM adjustments

Range of Motion
Fully Clockwise (CW) = MAX
Requires the most Body Sensor movement to initiate cursor travel. **Best setting for beginners** if able.
Fully CCW = MIN
Requires least Body Sensor movement to initiate cursor travel. For users with ALS & others with minimal movement
Can be difficult to control as is **extremely sensitive**

Midrange 6 - 8
Most common user settings

Click	Drag	Opti
L&R	R	= 0
L	L	= 1

Length of Sound
Determines clicks

Option 0
Left Click = Quick sound
Right Click = Long sound
Right Drag = Extended sound

Option 1
Left Click = Quick sound
Left Drag = Extended sound

DIP Switch
Selects Clicking Options
(On bottom of unit)

Default Settings
Range of Motion—MAX
DIP Switch—Single
(On bottom of unit)