


LaZee Mouse CURSOR

Quick Start

1 Connect Device*

* See back of sheet for additional info

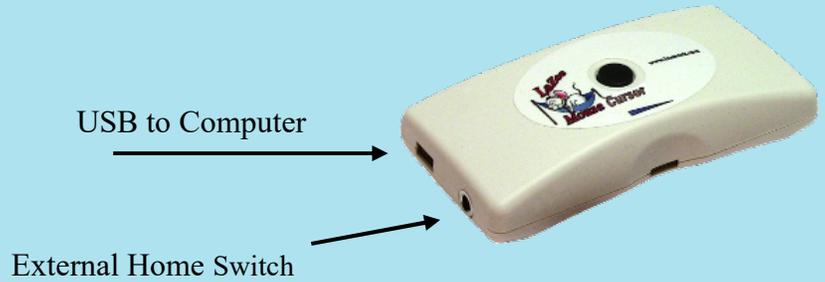

2 Position unit

Place LaZee Mouse anywhere desired.
(hat, other headset or.....)

OR



With provided headset
Place on head

Cables to Right side

Get Comfortable in front of screen
then

Press "Home"



3 Tilt to move cursor

Proportional Speed

- Tilt slightly to move SLOWLY
- Tilt farther to move Faster

Move Left



Tilt Left
(Lateral tilt)

Move Up

Tilt Head Up

Move Down

Tilt Head Down

Stop

Return to HOME position

Move Right



Tilt Right
(Lateral tilt)

Rotation of head
does NOT move
cursor.

4 Clicking

By External Switches

This unit provides no clicking, however, it is fully compatible with any hardware or software product that does.

* See back of sheet for additional info

LaZee Mouse CURSOR - Controls

