

LaZee Mouse SWITCH

Quick Start

1 Connect Device*

* See back of sheet for additional info

USB to Computer →

To Sip/Puff or other Switch(es) →



2 Position unit

Place LaZee Mouse anywhere desired.
(hat, other headset or.....)

OR



With provided headset
Place on head

Cables to Right side

Get Comfortable in front of screen
then

Press "Home"



3 Tilt to move cursor

Proportional Speed

- Tilt slightly to move SLOWLY
- Tilt farther to move Faster

Move Left



Tilt Left
(Lateral tilt)

Move Up

Tilt Head Up

Move Down

Tilt Head Down

Stop

Return to HOME position

Move Right



Tilt Right
(Lateral tilt)

Rotation of head
does NOT move
cursor.

4 Clicking

By External Switches

Option 1 - Two Switches control R & L buttons directly

OR

Option 0 - Single Switch controls L & R Clicks & Right Drag:

LaZee Mouse determines function by **length of switch closure**

Left Click - Quick closure

Right Click ~1 sec closure

Right Drag - Extended closure

* See back of sheet for additional info

LaZee Mouse SWITCH - Controls

Home
Calibrates the unit's position

Activate HOME when
in position

USB to Computer



Switch Input (for Clicking)



Adapter

Two Mono Plugs

Separate left & right inputs
require adapter (shown above)


3.5mm Stereo Plug

Sip/Puff (or other dual switches)
with a single 3.5mm stereo plug;
connects direct to unit.

One Mono Plug

One single external switch;
connects direct to unit
(Requires adapter to use Option 1)

DIP Switch Selects Clicking Options

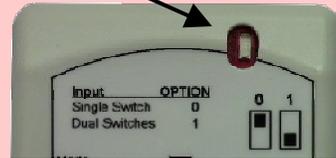

DUAL Switches = Option 1

Two switches—direct control of
all clicks and drags

Single Switch = Option 0

Length of closure determines:

- Left Click = Quick Closure
- Right Click = Long Closure
- Right Drag = Extended Closure